


**ZOLL AED 3™**

Wi-Fi Setup to PlusTrac

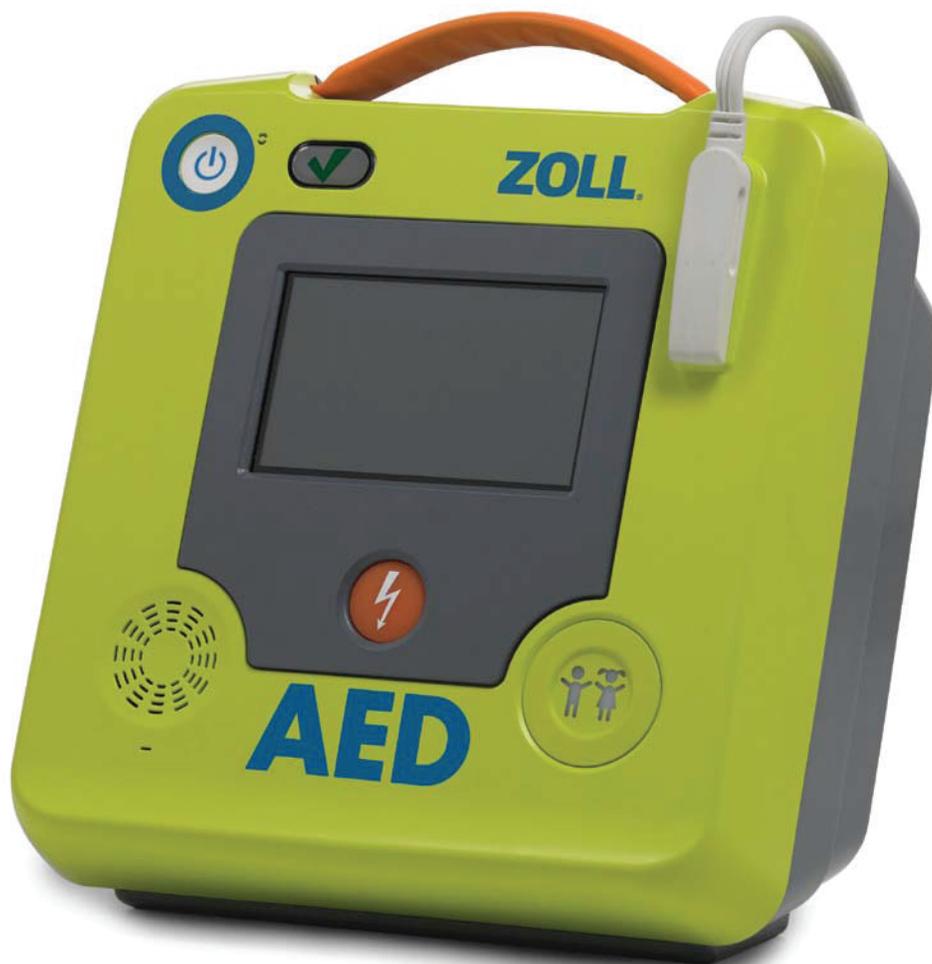

**PlusTrac**


## STEP 1 Turn on ZOLL AED 3 Defibrillator

Press and hold the On/Off button in the upper left for 30 seconds




On/Off button

On/Off button

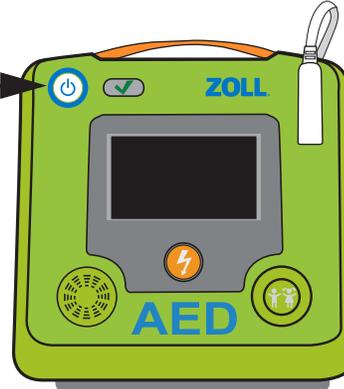

## STEP 2 Select Configuration Mode


Press the button with the picture of gears

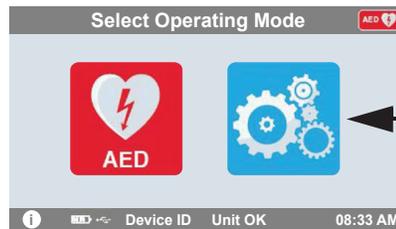

Configuration Mode

## STEP 3 Once in Configuration Mode Press Unlock

Click the orange button with the picture of the lock

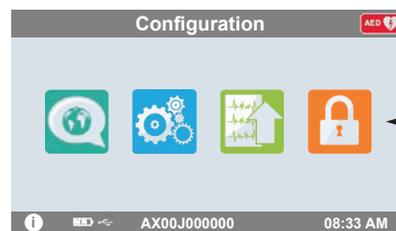

Unlock


## STEP 4 Enter the Supervisor Passcode

Default code is 1 2 3 4 5 6 and select OK



## STEP 5 Enter the Wi-Fi Setup menu

Press the icon with the ZOLL AED 3

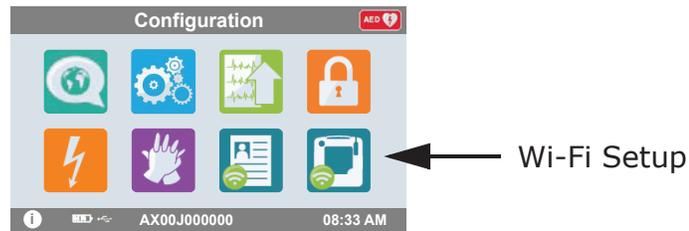


## STEP 6 Select the Wi-Fi Level

Press Quick Setup button



## STEP 7 Once in Setup Mode Press Wi-Fi icon

Click the green button to search for available networks

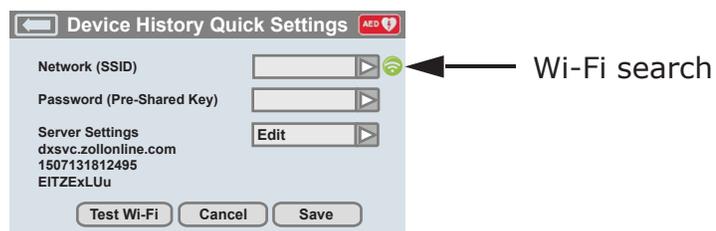


## STEP 8 Wait for the AED 3 to find signal

Search may take a minute



## STEP 9 Choose the correct network from the list

Press OK if Network is found, if no network type the name

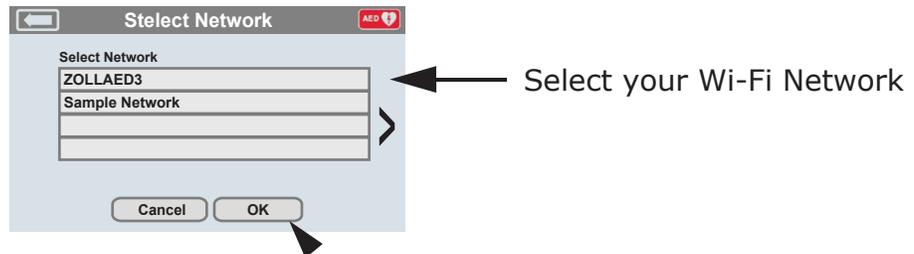

## STEP 10 Enter the Password for your Network

Type the pass key into the box using the keyboard and press OK



## STEP 11 Testing your Wi-Fi Network

After Setup is complete you can test your connection




## STEP 12 When Test is Complete you are READY!

Your AED 3 can now communicate to PlusTrac




## STEP 13 Log on to the PlusTrac2.com website

Enter the User Name (email address) and Password provided (new customers will need to complete "Activation" and registration)

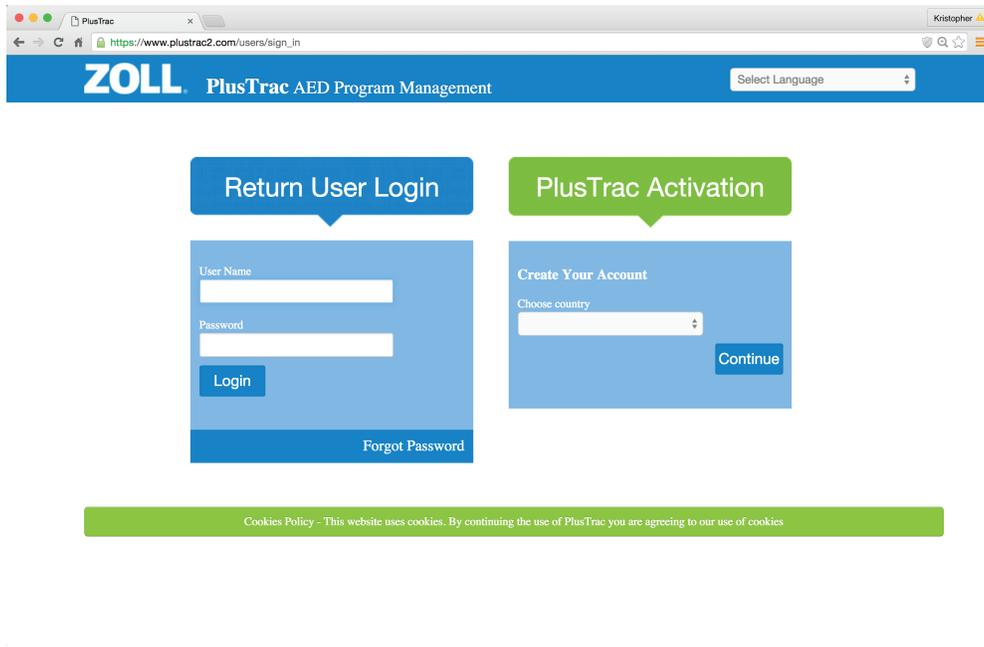


## STEP 14 Accessing your Device History

Press the Equipment Tab on the left Menu Bar

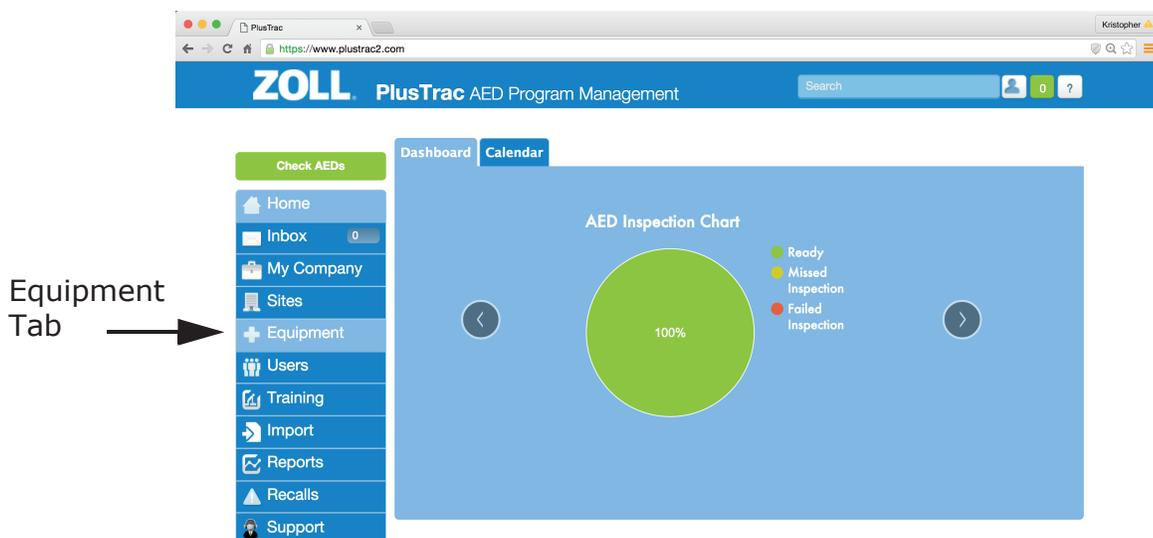

## STEP 15 Viewing the AED 3 information card


Click the Blue Arrow to Expand the Card and review information

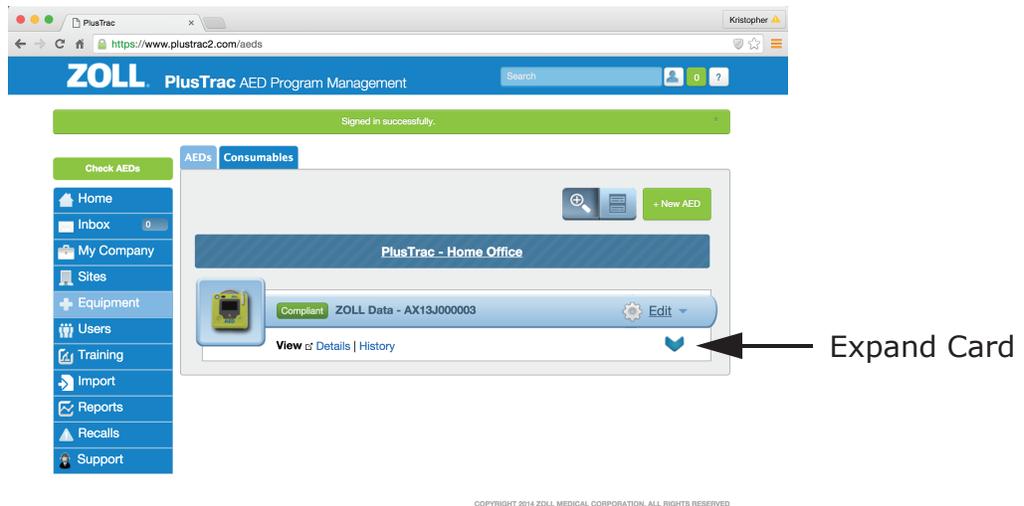

The screenshot shows the ZOLL PlusTrac AED Program Management interface. The main content area displays a card for a ZOLL Medical - AED 3 device with serial number AX13J000003. The card is currently collapsed. A blue arrow points to the 'Expand Card' button, which is a blue downward-pointing chevron icon located at the bottom right of the card. The interface includes a sidebar with navigation options like Home, Inbox, My Company, Sites, Equipment, Users, Training, Import, Reports, Recalls, and Support. The top navigation bar shows 'AEDs' and 'Consumables' tabs.

## STEP 16 Selecting the Device History

Press the Blue text that reads "History" on the Card

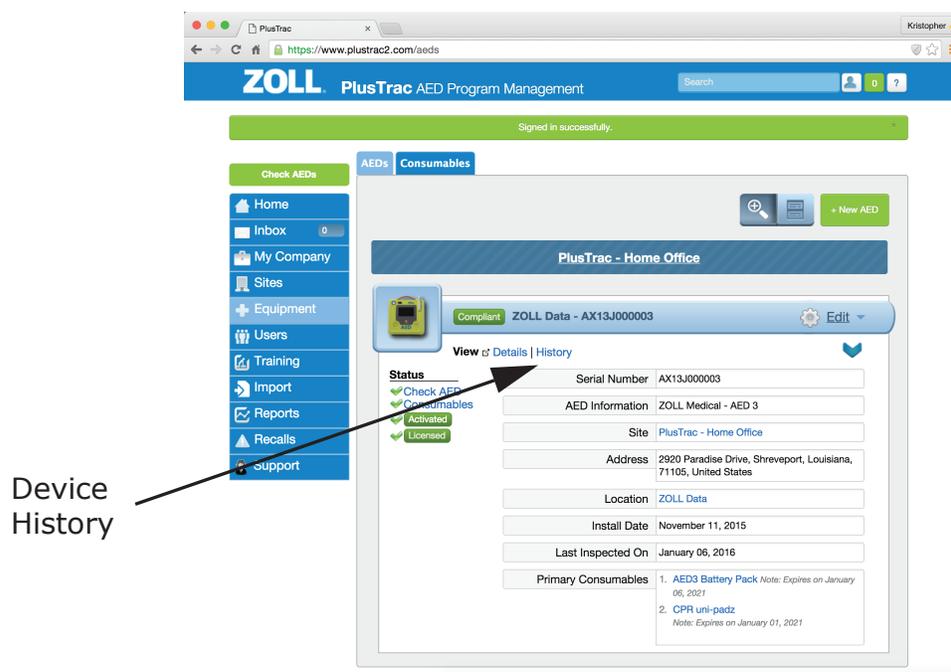

The screenshot shows the ZOLL PlusTrac AED Program Management interface with the AED 3 information card expanded. The card header includes a 'View of Details | History' link. A blue arrow points to the 'History' link. The card content displays various fields and a status section. The status section includes a 'Check AED' button and a 'Consumables' section with 'Activated' and 'Licensed' indicators. The main content area shows fields for Serial Number, AED Information, Site, Address, Location, Install Date, Last Inspected On, and Primary Consumables.

Device History

Field	Value
Serial Number	AX13J000003
AED Information	ZOLL Medical - AED 3
Site	PlusTrac - Home Office
Address	2920 Paradise Drive, Shreveport, Louisiana, 71105, United States
Location	ZOLL Data
Install Date	November 11, 2015
Last Inspected On	January 06, 2016
Primary Consumables	1. AED3 Battery Pack Note: Expires on January 06, 2021 2. CPR uni-padz Note: Expires on January 01, 2021

## STEP 17 Viewing the AED 3 communication

You will see a list of all the information sent from the AED 3. Select the event you want to view by clicking the Blue Text

The screenshot shows the PlusTrac AED Program Management interface. The main content area displays a table of events:

Who	What	When
Kristopher Edgell	ZOLL Medical - AED 3 - AX13J000003 has been updated	January 06, 2016
Kristopher Edgell	ZOLL Medical - AED 3 - AX13J000003 has been updated	January 06, 2016
Kristopher Edgell	PlusTrac - Home Office - ZOLL Medical - AED 3 - AX13J000003 (failed)	January 06, 2016
Kristopher Edgell	ZOLL Medical - AED 3 - AX13J000003 has been updated	January 06, 2016
System	ZOLL Broomfield - ZOLL Medical - AED 3 - AX13J000003 (failed)	January 06, 2016
System	ZOLL Broomfield - ZOLL Medical - AED 3 - AX13J000003 (failed)	December 07, 2015
System	ZOLL Broomfield - ZOLL Medical - AED 3 - AX13J000003 (failed)	December 04, 2015
System	ZOLL Broomfield - ZOLL Medical - AED 3 - AX13J000003 (failed)	November 30, 2015
System	ZOLL Broomfield - ZOLL Medical - AED 3 - AX13J000003 (failed)	November 24, 2015
System	ZOLL Broomfield - ZOLL Medical - AED 3 - AX13J000003 (failed)	November 24, 2015
System	ZOLL Broomfield - ZOLL Medical - AED 3 - AX13J000003 (failed)	November 23, 2015
Ailyn Scott	ZOLL Medical - AED 3 - AX13J000003 has been created	November 11, 2015

An arrow points to the event 'PlusTrac - Home Office - ZOLL Medical - AED 3 - AX13J000003 (failed)' with the label 'Self Test Results'.

## STEP 18 Understanding the Device History

PlusTrac record the communication of the AED 3 and will send Email alerts for any actions that need to be taken

The screenshot shows the PlusTrac AED Program Management interface displaying the details of a specific event:

- Who:** Kristopher Edgell
- What:** ZOLL Medical - AED 3 - AX13J000003 has been updated
- When:** January 06, 2016
- Changes:** Last Inspection Date changed from 2016-01-06 19:27:24 UTC to 2016-01-06 20:09:42 UTC